DEUTSCH

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WICHTIG!

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Adapters for electrosurgery

IMPORTANT!

Please read all information carefully.

These instructions for use do not replace the user manual of the electrosurgical unit used! Read the user manual of the electrosurgical unit and ask Erbe or your distributor in case of doubt!

1 Normal Use

These products connect various instruments to Erbe units.

2 Maximum electrical capacity

The maximum electrical capacity of these products is:

- **4 kV_p**: 20183-014, 20183-026, 20183-028, 20183-029, 20183-034, 20183-035, 20183-043, 20183-047.
- **800 V_p**: 20183-010, 20183-017, 20183-025, 20183-053.
- **500 V_p**: 20183-019, 20183-055, 20183-065.

3 Safety instructions

WARNING!

This product may be used only by trained medical staff, who have been shown how to use it according to the instructions for use.

Check the product before every occasion of use!

If damaged, do not use this product!

Do not use in the presence of combustible or explosive materials!

Never lay this product on the patient or in his/her direct vicinity.

Protect this product from any form of mechanical damage! Do not throw! Do not use force!

Erbe Elektromedizin expressly warns against modifying the product. Any modification exempts Erbe Elektromedizin from any and all liability.

4 How to use

The products must be cleaned and disinfected before using them for the first time!

The product must be fully inserted in the corresponding socket. The limit stop must be clearly felt. Pull the product out of the socket, keeping it straight. Transverse forces can otherwise damage the socket.

The adapter can remain in the device after use.

5 Cleaning and disinfection

This product is not suitable for mechanical cleaning and disinfection!

The adapter can remain in the device after use and be cleaned and disinfected together with it. The adapter is subject to the same reprocessing recommendations as for the device being used. Please read the User Manual for your device.

It is essential to follow the cleaning agent and disinfectant manufacturers' instructions.

Do not sterilize!

USA

Adapters for electrosurgery

IMPORTANT!

Please read all information carefully.

These notes on use do not replace the user manual of the electrosurgical unit (ESU) used! Read the user manual of the electrosurgical unit and ask Erbe or your distributor in case of doubt!

1 Normal Use

The adapters are used to connect various instruments to Erbe electrosurgical generator units.

2 Maximum Electrical Capacity

The maximum electrical capacity of this product is:

- **4 kV_p**: 20183-014, 20183-026, 20183-028, 20183-029, 20183-034, 20183-035, 20183-043, 20183-047.
- **800 V_p**: 20183-010, 20183-017, 20183-025, 20183-053.
- **500 V_p**: 20183-019, 20183-055, 20183-065.

3 Safety Instructions

WARNING!

This product may only be used by medical personnel who have been trained per these instructions.

Before each use, check that the adapter is intact and any connecting cable is

insulated. Verify that there are no signs of physical damage. Check the activation functions.

If damaged, do not use this product!

Do not use in the presence of combustible or explosive materials!

Never lay this product on the patient or in his/her direct vicinity.

Protect this product from any form of mechanical damage! Do not throw! Do not use force! If there is a cable, and unless stated otherwise, do not kink it or wrap it around the accompanying product!

Erbe Elektromedizin expressly warns against modifying the product. Any modification exempts Erbe Elektromedizin from any and all liability.

4 How to Use

Clean/disinfect this product before using it for the first time.

Connect the adapter to the electrosurgical unit by fully inserting it straight into the corresponding socket of the equipment. The stop limit must be clearly felt. To remove the adapter, pull it straight out of the socket (Note: Transverse forces may damage the unit's socket).

The adapter can remain attached to the unit after use.

5 Cleaning, Disinfection

This product is not suitable for mechanical cleaning and disinfection!

The adapter can remain attached to the unit after use and be cleaned and wipe disinfected together with it. In this case, please follow the same instructions for cleaning and disinfection of the unit in its User Manual.

It is essential to follow the cleaning agent and disinfectant manufacturers' instructions.

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