

## EIP 2

The Erbe EIP 2 irrigation pump –  
for a clear view of the target tissue



# Greater safety in endoscopic procedures – a clear view for diagnosis and therapy

*The EIP 2 irrigation pump flushes the target tissue clear of feces, food remnants, clots or blood. EIP 2 can be activated for irrigation during diagnosis and therapy, so that an additional irrigation catheter is no longer required.*



*After irrigation of the organ wall, increased visibility of the target tissue and improved electrosurgical application.*

## The advantages of irrigation

In electrosurgical cutting and coagulation, adhesion is reduced on moist, irrigated tissue. In addition, irrigation also allows sources of bleeding to be more effectively exposed, enabling efficient and targeted coagulation.

Erbe EIP 2 is activated either by using the footswitch or directly at the unit. The quick activation buttons on the display allows the user to select between three different irrigation levels – 30 %, 50 % or 80 % as standard, which can be modified and stored individually.

Using an irrigation adapter, the tubing set can be directly connected to the instrument channel of the endoscope. As a result, irrigation can be carried out while an instrument is inserted, without changing the instrument.



# Technical data

## EIP 2

No. 10325-000	Type of pump	Peristaltic roll pump
	Type of operation	Continuous

## Pump operation

Activation	Regulation via footswitch or control panel of the unit
Flow rate	Variable adjustment via the control panel of the EIP 2. 3 quick activation buttons (pre-set values of 30, 50 and 80 %)
Maximum flow rate with tubing set	500 ml/min corresponds to 100%

## Mains connection

Line voltage	220–240 V~
Power frequency	50 Hz

## Classification

Protection class	Classification according to EN IEC 60601-1: Class I, Type CF
Medical product classification	Classification according to EU Directive 93/42/EEC: IIa

## Accessories

One pedal footswitch	20325-000
Tubing set for irrigation forceps	20325-020
Tubing set for flexible endoscope with Luer Lock connection	20325-001
Irrigation adapter for Olympus, Fujifilm G5, STORZ	20325-003
Irrigation adapter for Fujifilm	20325-004
Irrigation adapter for PENTAX	20325-005



Erbe Elektromedizin GmbH  
Waldhoernlestrasse 17  
72072 Tuebingen  
Germany

Phone +49 7071 755-0  
Fax +49 7071 755-179  
info@erbe-med.com  
erbe-med.com